# B.Well

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INFRARED FOREHEAD THERMOMETER WF-2000

Measurement  
HEAD  
temperature

Measurement  
ENVIRONMENT  
temperature

### PROBLEMS:

Drawing 1

Drawing 2

Drawing 3

Drawing 4

Drawing 5


Drawing 6

Drawing 7

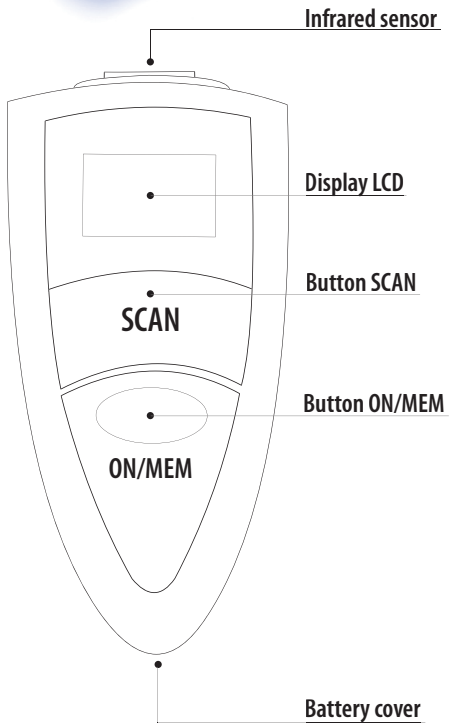Drawing 8

**Please read the instructions and warnings given in this handbook carefully before use, and always keep it in a safe place.**

Your new forehead thermometer detects body temperature in an absolutely precise and painless way.

It is ideal to carry out measurements on babies, even while they are sleeping.

The new infrared measurement technology offers a greater reliability and above all a longer life of the instrument.

It is equipped with a wide LCD display for an easy reading.

It has also an acoustic signal for completed measurement of the body temperature and of automatic switching-off after one minute of inactivity, which allows to extend the duration of the batteries.

It is really easy to be used and allows the measurement of body and room temperature.

It measures temperature in Celsius (C°) and Fahrenheit (F°) degrees.

It stores in memory the last 25 measurements performed to evaluate eventual variations.

**NB: the temperature shown on the display is equal to the oral temperature, i.e. temperature measured on the forehead is translated as if it were measured in the mouth.**

Manufacturer: B.Well Limited

758 Great Cambridge Road, Enfield, Middlesex, EN1 3PN. United Kingdom

## ■ GENERAL WARNINGS

---

**1. Self-measuring means control, it does not mean diagnosis or treatment.**

**Any unusual levels should always be discussed with your doctor.**

**Under no circumstances must you change the dose of medicines prescribed by your Doctor.**

**2. The forehead thermometer is a very sensitive instrument.**

Please handle it with care and avoid knocks.

Do not press, bend, drop or break the thermometer into small pieces.

3. Keep it out of the reach of children.

4. The thermometer must not be exposed to extreme temperatures, direct sunlight or sources of heat.

It must be kept in a cool, dry and clean place.

5. Do not touch the probe with your fingers.

Do not dismantle the thermometer.

6. Do not immerse the thermometer in water or other liquids.

7. The forehead thermometer must not be used for rectal, oral or armpit temperature reading.

8. The precision of the measurement can be jeopardized by: great perspiration of the forehead, use of vasoconstrictive drugs and skin irritations.

9. Clean the temple where the probe will lean before performing the measurement.

10. Measure the temperature far from direct sunlight and wind.

11. This thermometer has to be used to measure temperature on forehead (with skin contact) as shown on the cover page.

## ■ A HIGH TEMPERATURE

A high temperature should be considered a symptom and not an illness. Usually it is the sign that our organism is fighting against an infection.

In fact, it represents a defensive reaction of our organism that is characterised by an increase in body temperature above  $37^{\circ}\text{C}$  and by an increase in heartbeat and breathing frequency.

The normal temperature inside our organism is around  $37.5^{\circ}\text{C}$ , which is the value read with the rectal method, the temperature in the mouth is lower by about  $0.5^{\circ}\text{C}$  ( $37^{\circ}\text{C}$ ), while that under the armpit is lower by  $1^{\circ}\text{C}$  ( $36.5^{\circ}\text{C}$ ).

Therefore it is very important that the temperature is correctly measured using the most effective method, and preferably not in those moments when body temperature could be higher (on waking, after meals).

The temperature of a healthy person is also affected by the following factors:

- The person's individual level (individual metabolism).
- Age (body temperature is higher in babies and small children and falls with increasing age. Greater temperature fluctuations occur in children faster and more often.).
- Clothing.
- The outside temperature.
- The time of day (Body temperature is lower in the morning and rises as the day goes on).
- Preceding physical activity.
- Measuring method.
- Phase of the menstrual cycle.

You should remember that skin temperature is easily affected by the room temperature; in fact, taking the temperature under the armpit is much less reliable with respect to oral and rectal readings.

However, the temperature that is read on the forehead is extremely reliable and close to the real body temperature, as just a few mm from the skin on the forehead the temporal artery passes which carries the blood to the brain.

This means that your new infrared forehead thermometer measures the temperature of the blood in the artery that passes underneath the skin. This is a highly appreciated method as it is very simple, precise and reliable.

The graph below shows a list of the average "normal" temperatures.

**Please remember, that the thermometer must be in the room where the temperatures is to be taken at least half an hour beforehand, while the person must be present at least 10 minutes beforehand.**

**However, we recommend that you learn to recognise your**

**normal body temperature when you are well, to correctly interpret the different readings.**

#### **MEASURING METHOD NORMAL AVERAGE**

ARMPIT 36.5°C

**ORAL 37°C**

RECTAL 37.5°C

EAR 37°C

These different thermometers are suitable for measuring the temperature in different parts of the body: forehead thermometer: measurement only on the forehead, ear thermometer: measurement only in the ear, rod thermometer: rectal (conventionally in the back passage), axillary (under the arm) or buccal measurement (inside the cheek) and under the tongue (sublingual).

The temperature varies depending on the part of the body where it is measured. The variation can be between 0.2 – 1°C.

Temperatures measured with different thermometers cannot be compared with one another. Tell your doctor or take into account when diagnosing yourself what thermometer you used in what part of the body to measure the temperature.

#### **■ INSERTING/REPLACING THE BATTERY**

The battery has to be urgently replaced when on the display the symbol of “discharged battery” appears.

Before replacing the battery, switch off the thermometer to avoid damaging the mechanism.

- 1) extract the screw from the cover of the battery compartment.
- 2) open the cover and remove the battery
- 3) insert the new battery and place the positive pole upwards.
- 4) place the cover back on the battery compartment.
- 5) screw the screw in the cover.

Do not dispose used batteries as domestic waste, but deposit them in the proper collection points.

**■ INSTRUCTIONS FOR A CORRECT USE****MEASUREMENT OF THE FOREHEAD TEMPERATURE (HEAD)**

- 1) Make sure that the probe is clean and integral.
- 2) Clean the temple where the probe will lean.
- 3) Switch on the thermometer, by pressing the key "ON/MEM". On the display appear all the function symbols (self-test); after few seconds, the display shows the message "HEAD-C" and emits two beeps. At this point, the thermometer is ready for use.

4) Lean the probe on the right or left temple; press the key SCAN and keep it pressed, then slowly move the probe around the temple.

The thermometer emits beeps indicating that the measurement is taking place.

5) A longer and more intense acoustic signal indicates that the measurement has ended (Time necessary to perform the measurement ranges from 5 to 30 second according to the temperature of the person).

Should the temperature exceed 37.5°C, the thermometer emits a long beep followed by other three shorter beeps.

6) Release the key SCAN and then detach the probe from the temple; read the measured value on the display

**NB:** the temperature shown on the display is equal to the oral temperature, i.e. temperature measured on the forehead is translated as if it were measured in the mouth.

7) The device switches off automatically after 1 minute, if it is not used.

**■ MEASUREMENT OF THE ROOM TEMPERATURE (  )**

This thermometer allows the measurement of the room temperature.

1) Switch on the thermometer, by pressing the key "ON/MEM". On the display appear all the function symbols (self-test); after few seconds, the display shows the message "HEAD-C" and emits two beeps.

2) Keep the key "ON/MEM" pressed and then press the key SCAN.

The display shows the icon " " (HEAD disappears).

3) If you wish to measure the temperature on a precise point, press the key SCAN.

The thermometer measures the temperature in 0.6 seconds. If the SCAN key is kept pressed, temperature is updated continuously.

4) The device switches off automatically after 1 minute, if it is not used. When it is switched on again, the display shows the forehead temperature measurement function (HEAD).

## ■ CELSIUS/FAHRENHEIT

---

This thermometer can measure temperature both in Celsius and Fahrenheit degrees.

Each time you wish to change the measurement unit, proceed as follows:

1) With switched off thermometer, keep the key SCAN pressed without releasing it for some seconds, and then press contemporaneously the key ON/MEM until the displays shows °F (or °C).

2) The thermometer emits two beeps; at this point, release the key SCAN and the key ON/MEM.

Now the measurement unit has been changed.

Proceed with the measurement according to paragraph "INSTRUCTIONS FOR A CORRECT USE".

## ■ MEMORY FUNCTION

---

This device stores in memory the last 25 measurements performed to evaluate eventual variations. The measurements are stored automatically after that the thermometer has switched off.

Only forehead temperatures (HEAD) included between 22°C and 42.2°C (71.6°F and 108.0 °F) are stored.

After the first 25 measurements, the oldest data are erased automatically.

To call up saved measurements, switch on the thermometer, and press the key ON/MEM (Every time the key is pressed, saved data are

scrolled). On the upper part of the display appears the symbol "M" and before each stored data appears the corresponding number (1= most recent measurement - 25 oldest measurement).

The stored measurements are displayed for about 1 minute.

After about 1 minute from releasing the ON/MEM key, the device switches off automatically.

## ■ CLEANING AND MAINTENANCE

---

Before and/or after each use, clean the probe of the thermometer with a soft cotton cloth moistened with water or alcohol to assure accurate measurements and to avoid microbial contaminations. The probe is the most delicate part of the thermometer; pay utmost care during the cleaning.

Let the probe dry completely for about 30 minutes.

The thermometer body has to be cleaned with a soft and dry cloth.

THE BODY OF THE THERMOMETER HAS NOT TO BE WASHED WITH WATER.

Absolutely neither use abrasive products, nor plunge the thermometer in water or other liquids.

Store the thermometer in a dry and clean place away from direct sunlight.

## ■ TROUBLESHOOTING

---

### Drawing 1

Problem: The "HEAD" writing blinks: the mechanism is adjusting itself.

Solution: Await some seconds until the "HEAD" message stops to blink.

### Drawing 2

Problem: Discharged battery

Solution: Replace the battery

### Drawing 3

Problem: The thermometer detects rapid room temperature changes.

Solution: It is necessary to consider that the thermometer has to be in the room where the measurement is performed about half an hour before performing the measurement, while the person has to be in the room at least 10 minutes before. (Room temperature has to range between  $+10^{\circ}\text{C}$  and  $+40^{\circ}\text{C}$  / between  $50^{\circ}\text{F}$  and  $104^{\circ}\text{F}$ )

#### **Drawing 4**

Problem: Room temperature is not included between  $+10^{\circ}\text{C}$  and  $+40^{\circ}\text{C}$  / between  $50^{\circ}\text{F}$  and  $104^{\circ}\text{F}$ .

Solution: It is necessary to consider that the thermometer has to be in the room where the measurement is performed about half an hour before performing the measurement, while the person has to be in the room at least 10 minutes before. (Room temperature has to range between  $+10^{\circ}\text{C}$  and  $+40^{\circ}\text{C}$  / between  $50^{\circ}\text{F}$  and  $104^{\circ}\text{F}$ ).

#### **Drawing 5**

Problem: Error 5~9, the device does not work correctly.

Solution: Remove the battery, await 1 minute and then place it again in the proper compartment.

Should the error persist, contact your dealer.

#### **Drawing 6**

Problem: Temperature exceeds  $42.2^{\circ}\text{C}$  –  $108^{\circ}\text{F}$ .

Solution: Verify the integrity of the probe and perform a new measurement.

#### **Drawing 7**

Problem: Temperature is lower than  $22^{\circ}\text{C}$  –  $71.6^{\circ}\text{F}$ .

Solution: Verify that the probe is clean and perform a new measurement.

#### **Drawing 8**

Problem: The device blocks on self-test.

Solution: Replace the battery

## ■ TECHNICAL DATA

---

 Caution: check the instructions for use

 BF Type device

Battery: 1 x CR2032LI

Display: liquid crystals

Switching-off: automatic, after one minute, if not used.

Measuring range:  $-22^{\circ}\text{C}$  and  $80^{\circ}\text{C}$  ( $-7.6^{\circ}\text{F}$  and  $176.0^{\circ}\text{F}$ ).

Resolution:  $0.1^{\circ}\text{C}$

Operation conditions:  $10^{\circ}\text{C}$  to  $40^{\circ}\text{C}$

Storage conditions:  $10^{\circ}\text{C}$  to  $50^{\circ}\text{C}$

Tolerance: less than  $22^{\circ}\text{C}$ ;  $\pm 2.0^{\circ}\text{C}$

From  $22^{\circ}\text{C}$  to  $42.2^{\circ}\text{C}$ ;  $\pm 0.3^{\circ}\text{C}$

More than  $42.2^{\circ}\text{C}$ ;  $\pm 2.0^{\circ}\text{C}$

## ■ WARRANTY

---

This appliance is warranted for a period of 1 year from the date of purchase as shown by the **dealer's stamp and signature and by the purchaser's receipt for tax purposes which must be kept attached to this warranty.** The warranty covers only any faults in manufacturing and does not cover accidental damage, incorrect use, negligence or misuse of the product.

Use only the accessories provided; the use of accessories other than the approved ones can make your warranty void. Never open the appliance; in the event of any opening or tampering the warranty will automatically become invalid.

The warranty does not cover either the parts subject to wear and tear or the batteries (if provided). The warranty expires 1 year from the date of purchase; after this period any technical service shall be charged. Information regarding technical service either under the terms of this warranty or to be paid for can be obtained by contacting [inform@b-well.ru](mailto:inform@b-well.ru).

No compensation of any kind is owed for repairs or substitutions of products under the terms of this guarantee. In the event of faults, go back to the retailer; DO NOT send the appliance directly to B.Well.

Use in public places is not allowed. The manufacturer shall not be-

liable for any damages to people, property or pets, either directly or indirectly, caused by the non compliance with the instructions laid down in the instruction manual, in particular the warnings regarding the installation, use and maintenance of the appliance. It is B.Well's policy to improve its products constantly; B.Well therefore reserves the right to change its products, either partially or totally, without previous notification and according to the manufacturing requirements, without either B.Well or its retailers being in any way liable for it.

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