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Mi, **Momert Zrt.**

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Manual Instruction NASAL ASPIRATOR Model 1723

Thank you for purchasing a Momert product. We hope you will be satisfied with our product through its service life.

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

CONTENTS:

- 1 Main unit
- 2 sizes of silicone tips (1 adult and 1 child)
- 1 instruction manual
- 2 spare silicone rings
- 2x1,5V AA alkaline batteries (included)

OPERATION, STORAGE AND TRANSPORTATION ENVIRONMENT INFORMATION

Operation environment:

Temperature: 5°C–40°C
Humidity: 15%~93%
Atmospheric pressure: 700-1060 hPa

Storage and transportation environment:

Temperature: -10°C-50°C
Humidity: ≤93%
Atmospheric pressure: 700-1060 hPa

IMPORTANT SAFETY INSTRUCTIONS:

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the

appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

- This device is to clean stuffy noses, do not use it in baby's mouth or ears.
- Always clean and dry thoroughly before each use and after each use.
- Do not insert the silicone tip into the nostril too deep.
- Do not use it on the baby for a long time. A few seconds can be enough.
- Do not use this device if there is any wound, or symptom of inflammation or bleeding at the nose.
- In case of infection, fever, nasal irritation or haemorrhage, please stop using this device and consult your physician.
- This device is restricted for personal use.
- The silicone tip is the applied part.
- Duty cycle: 30 seconds ON / 2 minutes OFF
- Replace batteries when the suction power becomes weak during use.
- **Batteries are not included in warranty.**

FAILURE TO FOLLOW THE MANUFACTURER'S INSTRUCTIONS MAY LEAD TO REFUSAL OF WARRANTY REPAIR!

BATTERY WARNING:

1. Use alkaline batteries only.
2. Only batteries of the same or equivalent type as recommended should be used.
3. Do not mix alkaline, standard (carbon-zinc), or rechargeable (nickel-cadmium) batteries.
4. Do not use a mix of used and new batteries.
5. Batteries should be inserted with the correct polarity.
6. Exhausted batteries should be removed from the device.
7. Be sure that all contact surfaces are clean

before installing batteries.

8. There is a risk of explosion if the battery is replaced by an incorrect type.
9. Remove batteries if the device will not be used for a long period of time.

PARTS OF DEVICE (Fig. 1.):

1. Silicone tip for children
2. Collar
3. Button
4. Silicone tip for adult
5. Mucus cup
6. Silicone rings
7. Upper cover
8. Lower cover

INSTRUCTIONS FOR USE (Fig.2.):

1. Separate the upper cover away from the lower cover in a straight direction. Do not bend the covers by strong force as it can be slid open easily.
2. Insert 2x1,5V AA alkaline batteries with the correct polarity as shown.
3. Check to make sure that the two silicone rings are properly seated on the grooves. Close the covers until the "click" sound is heard.
4. Use the suitable silicone tip based on the size of user's nostrils.
5. Be sure to place the silicone tip securely on the mucus cup.
6. Gently insert the silicone tip into the nostril not too deep.
7. Press the button for a few seconds to collect mucus. Stop pressing the button when finished using the device and then the power will be stopped.

If the accumulated volume of mucus is nearly over the safe capacity of the mucus cup (about 4ml), follow the section of "Cleaning Instructions" to clean the device first. Do not use the device if the mucus level is nearly

over the opening channel (where the arrow points to in the picture).

CLEANING INSTRUCTIONS (Fig.3.):

1. Be sure that the upper and lower covers are closed firmly before cleaning the device.
2. Remove the silicone tip.
3. Hold the mucus cup with the part between thumb and index finger.
4. Bend the mucus cup downwards gently to remove it from the collar.
5. Remove the silicone rings from the collar for cleaning after several uses of the device. The silicone rings do not need to be cleaned every time.
6. Wash all 4 parts in warm soapy water. Rinse the collar under the tap. Dry the parts before assembling them.
7. Assemble all the parts as shown below.

- Properly align the positioning hole on the mucus cup.
- Be sure all the parts fit tightly together and line up the indication marks.

When the accumulated volume of mucus is over the safe capacity of the mucus cup during use of the device, if you see mucus dripping out of the air vent on the back side of the collar, please follow instructions below (Fig.4.):

1. Be sure that the upper and lower covers are closed firmly.
2. Remove the silicone tip and the mucus cup.
3. Have the collar immersed in clean water as shown. Press the button for 15 seconds and the inner pump will be cleaned.
4. The water flows in and out of the inner pump in the direction as shown.
5. Change the water and repeat step 3 several times if necessary.

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6. Remove the device from water once the cleaning is completed. Keep pressing the button for approximately 30 seconds until the inner pump is dried.
7. Dry the collar and reassemble the parts.

REPAIR AND MAINTENANCE:

An extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

ENVIRONMENTAL PROTECTION:

- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

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